Infinity Quick Operation Guide - Description of Parts and Controls of Equipment

Before using the equipment, please read the *User Manual Dental Line* carefully.

The purpose of this guide is to identify commands, parts, and instruments that may be available to the equipment, including optional items. This guide does not define the equipment configuration that may vary according to the customer's request.

In case of loss of the *User Manual* and/or *Operation Guide*, please contact us by e-mail export3@olsen.odo.br informing the serial number of your equipment.

Delivery System


Before Turning On the Equipment:

- 1 - Pressure Valve
- 2 - Water Tanks
- 3 - Protection Cover

- 1° - Check if the equipment is properly installed in accordance with the instructions in *chapter 6 (Installation)* of the *User Manual Dental Line*;
- 2° - Turn On the air compressor that supplies the equipment;
- 3° - Open the main water supply valve of the equipment;
- 4° - Turn On the equipment's power circuit breaker.

Remote Foot Pedal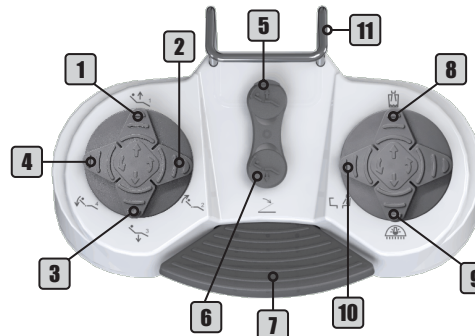

- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Spit Position
- 5 - Work Position
- 6 - Zero Position
- 7 - Progressive/Activation Pedal
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - Spittoon Bowl Flush
- 11 - Metal Handle

To save a work position:

- 1° - Press the Zero Position button;
- 2° - Adjust the backrest to the desired position;
- 3° - Adjust the seat to the desired position;
- 4° - Press the Work Position button for 5 seconds;


The equipment will emit 2 long beeps.
5° - Press command 1 (Seat Up) while the equipment beeps; The equipment will emit 2 short beeps to confirm the procedure;
6° - Press the Zero Position button.

Operating Light


- 1 - Control knob
 - 2 - Sensor
- 1° - To activate the operating light (F04) press the On/Off command on the command panel (F06-9/F12-9/F13-9/F14-9) or remote foot pedal (F03-10) to enable sensor and control;
- 2° - To turn On/Off the operating light pass the hand in front of the sensor (2) approximately 5 cm away or press the control knob (1);
- 3° - To change the light intensity hold your hand in front of the sensor (2) or turn the control knob (1) until the operating light shows the desired intensity.

Assistant Module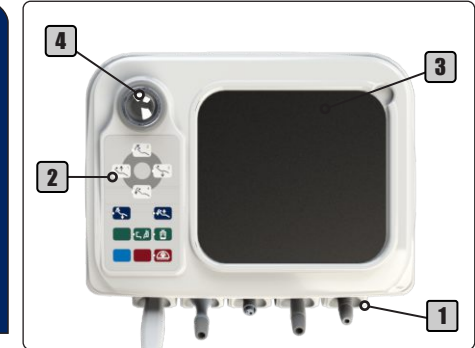

- 1 - Instruments Holder
- 2 - Assistant Module Commands
- 3 - Stainless Steel Tray
- 4 - Bicarbonate Reservoir

Assistant Module Commands

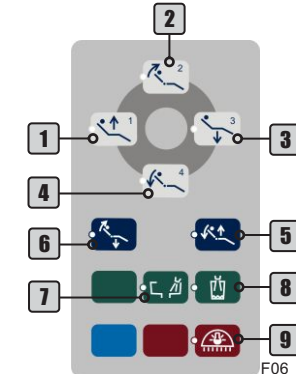- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Spit Position
- 5 - Working Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light

Chair Base

Turning On the Equipment:

- 1 - Emergency Stop Button
- 2 - Voltmeter
- 3 - Fuse Holder
- 4 - On/Off Switch of Chair


- 1° - Turn On the On/Off switch;
- 2° - Activate the "Seat Up" command on the pedal until the seat reaches its maximum height limit;
- 3° - Open the protection cover;
- 4° - Close the pressure valve;
- 5° - Make sure the water tanks are filled with water. If necessary, fill it with drinkable water up to the limit indicated in each tank and reattach them again. If you wish to place prophylactic products (low concentration);
- 6° - Open the pressure valve and check if the water tanks are well attached and free of leaks;
- 7° - Close the protection cover;
- 8° - Before starting to use the equipment, check the operation of all available instruments and controls and perform cleaning and sterilization of the instruments (*chapter 7 of the User Manual Dental Line - Cleaning and Disinfection*).

Observe the Scheduled Review system for your equipment.


Infinity Quick Operation Guide - Description of Parts and Controls of Equipment

Working Table

**Infinity Working Table
Mesa Infinity**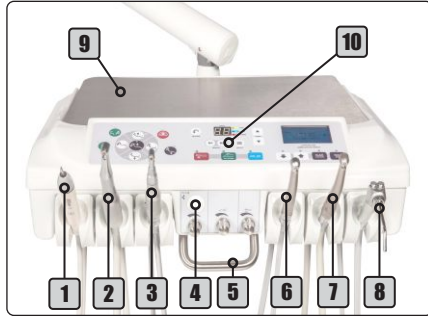

Mesa Cross Flex

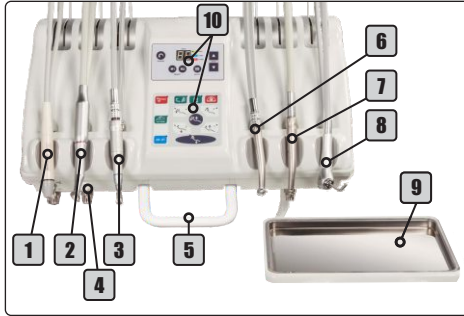
- 1 - Scaler with LED Light
- 2 - Coupling for Brushless Electrical Micromotor
- 3 - Coupling for Pneumatic Micromotor
- 4 - Instruments Control Panel
- 5 - Capacitive Handle
- 6 - Coupling for High-speed Handpiece
- 7 - Coupling for Fiber Optics Handpiece
- 8 - 3-Way Syringe
- 9 - Stainless Steel Tray
- 10 - Command Panel

Only move the working table through the capacitive handle

The Infinity working tables can present some configurations of command panels, varying according to available instruments.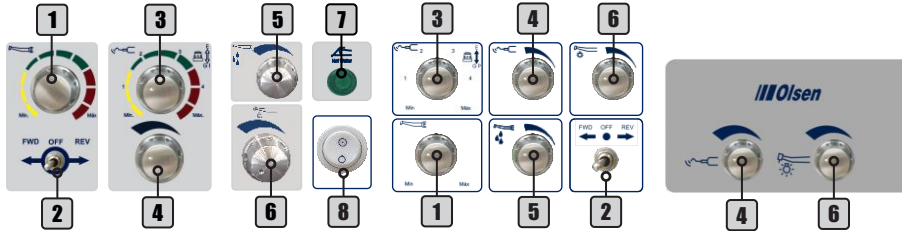

- 1 - Electrical Micromotor Speed Control
- 2 - Electrical Micromotor Direction Control
- 3 - Scaler Power Control and General Function Selector - Perio/Endo
- 4 - Scaler Water Control
- 5 - Brushless Micromotor Water Control
- 6 - Optical Fiber Water Control
- 7 - Thermo Comfort LED On/Off
- 8 - Fiber Optics Light On/Off


Flex Arm

The **Flex Arm** features 2 articulations (1) and pneumatic lock that are disabled by touch in the capacitive handle (2), allowing the working table movement.


Command Panel

Blue Touch Panel Models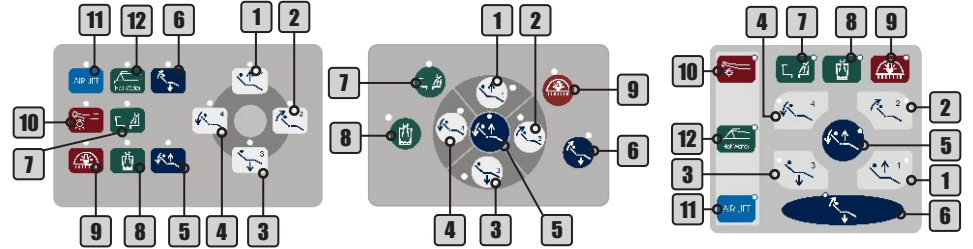



Blue Touch Panel - X-Ray Viewer

**Command Panel
Chronolub System**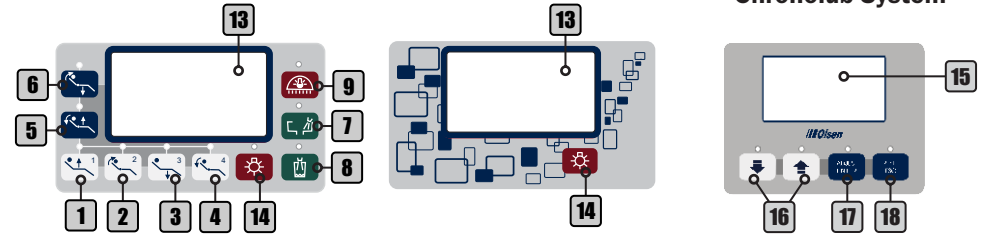

**Command Panel -
Brushless Electrical Micromotor**

**Command Panel
Bien Air Micromotor (Display Optional)**

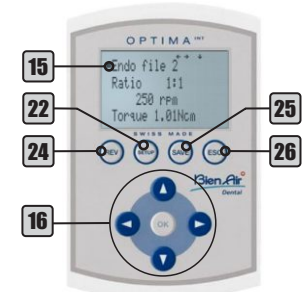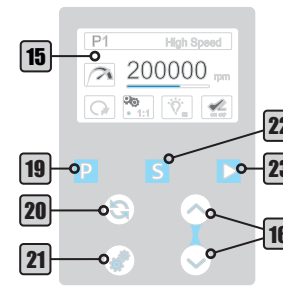- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down
- 5 - Work Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - On/Off Optical Fiber Light
- 11 - On/Off Air Jet System
- 12 - On/Off Thermo Comfort System
- 13 - X-Ray Viewer

- 14 - On/Off X-Ray Viewer
- 15 - Display
- 16 - Adjustment Keys
- 17 - Adjust/Enter Key
- 18 - Reset/Esc Key
- 19 - Program
- 20 - Direction
- 21 - Gear Ratio
- 22 - Setting/Setup
- 23 - Manual Start
- 24 - REV
- 25 - SAVE
- 26 - ESC

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