# Logic Quick Operation Guide - Description of Parts and Controls of Equipment

Before using the equipment, please read the *User Manual Dental Line* carefully.


The purpose of this guide is to identify commands, parts, and instruments that may be available to the equipment, including optional items. This guide does not define the equipment configuration that may vary according to the customer's request.

In case of loss of the *User Manual* and/or *Operation Guide*, please contact us by e-mail [export3@olsen.odo.br](mailto:export3@olsen.odo.br) informing the serial number of your equipment.

## Delivery System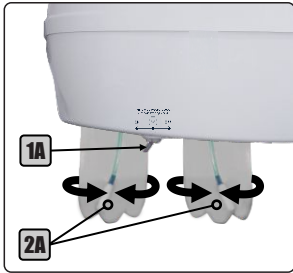

### Before Turning On the Equipment:


- 1A - Water Tank
- 2A - Pressure Valves

- 1° - Check if the equipment is properly installed in accordance with the instructions in *chapter 6* of the *User Manual Dental Line*;
- 2° - Turn On the air compressor that supplies the equipment;
- 3° - Open the main water supply valve of the equipment;
- 4° - Turn On the equipment's power circuit breaker.

### Turning On the Equipment:

- 1B - On/Off Switch with LED Indicator
- 2B - Fuse Holder
- 3 - Emergency Stop Key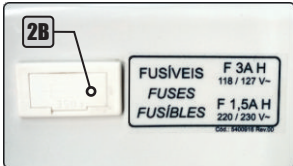


- 1° - Turn On the On/Off switch;
- 2° - Activate the "Seat Up" command on the chair base until the seat reaches its maximum height limit;
- 3° - Close the pressure valve;
- 4° - Remove the water tank and make sure it filled with water. If necessary, fill it with drinkable water up to the limit indicated in tank and reattach it again. If you wish, to place prophylactic products (low concentration);
- 5° - Open the pressure valve and check if the water tank is well attached and free of leaks;
- 6° - Before starting the use equipment, verify the operation of all instruments and commands available and perform the cleaning and sterilization of the instruments.

## Chair Base

## Remote Foot Pedal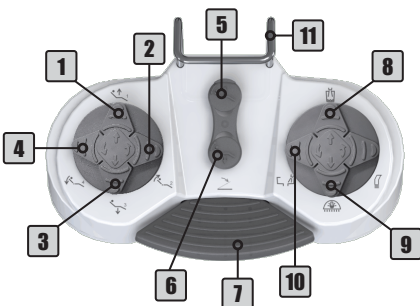


- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Spit Position
- 5 - Working Position
- 6 - Zero Position
- 7 - Progressive/Activation Pedal
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - Spittoon Bowl Flush
- 11 - Metal Handle

- To save a work position:**
- 1° - Press the Zero Position button;
  - 2° - Adjust the backrest to the desired position;
  - 3° - Adjust the seat to the desired position;
  - 4° - Press the Work Position button for 5 seconds;

- The equipment will emit 2 long beeps.
- 5° - Press command 1 (Seat Up) while the equipment beeps; The equipment will emit 2 short beeps to confirm the procedure;
  - 6° - Press the Zero Position button.

## Operating Lights

### Reflector Concept LED Plus

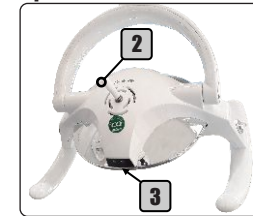
### Reflector Premium LED

- 1 - Control Knob
  - 2 - Control Rod
  - 3 - Sensor
- 1° - To activate the Operating Light, press the Operating Light On/Off button at the base of the chair (section Commands - 7), this action also enables the sensor (3) and the control button/ rod (2);


- 2° - To turn on or off the Concept Plus and LED Premium Operating Light, pass your hand in front of the sensor (3) at approximately 5 cm distance or hold the control rod (2), in the case of Concept Plus, or turn the control button (1), in the case of the Premium LED spotlight;
- 4° - To change the lighting intensity of the Concep Plus and Premium LED reflectors, keep your hand in front of the sensor (3) or hold the control knob (1), in the case of the Premium LED Operating Light.

## Assistant Module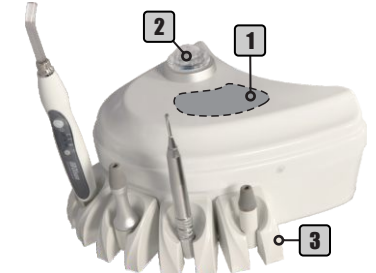

- 1 - Assistant Module Commands
- 2 - Sodium Bicarbonate Reservoir
- 3 - Instruments Holder



### Assistant Module Commands

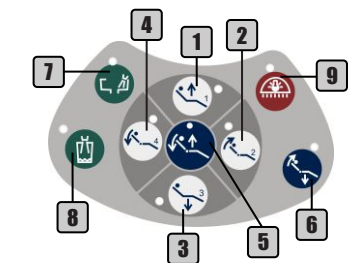- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Spit Position
- 5 - Working Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light

# Logic Quick Operation Guide - Description of Parts and Controls of Equipment

Working Table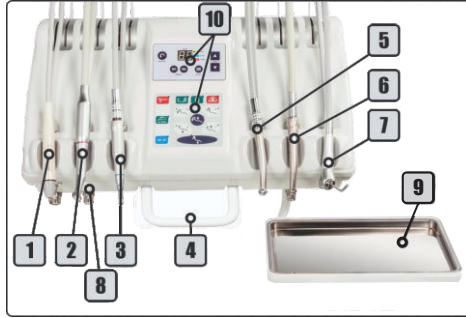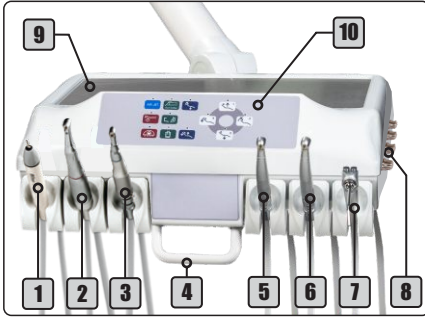


- 1 - Scaler with LED Light
- 2 - Coupling Micromotor
- 3 - Coupling Micromotor
- 4 - Capacitive Handle
- 5 - Coupling for High-speed Handpiece
- 6 - Coupling for Optical Fiber Handpiece
- 7 - 3-Way Syringe
- 8 - Instruments Control Panel
- 9 - Stainless Steel Tray
- 10 - Command Panel

The Logic working tables can present some configurations of command panels, varying according to available instruments.


**Only move the working table through the capacitive handle**

Command Panel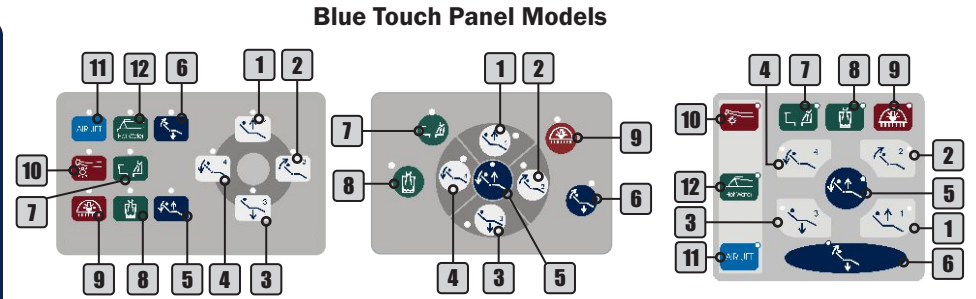

Blue Touch Panel - X-Ray Viewer


Command Panel Chronolub System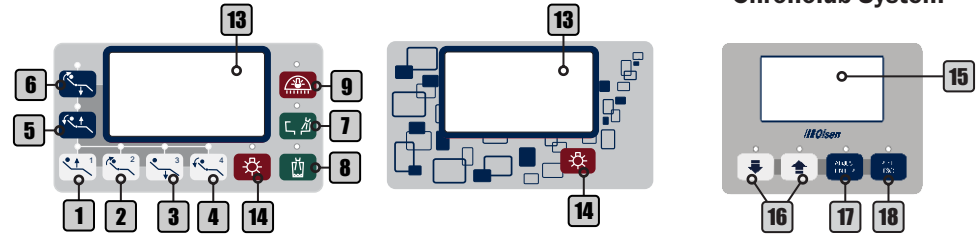

Command Panel - Brushless Electrical Micromotor

Command Panel Bien Air Micromotor (Display Optional)

Control Panel

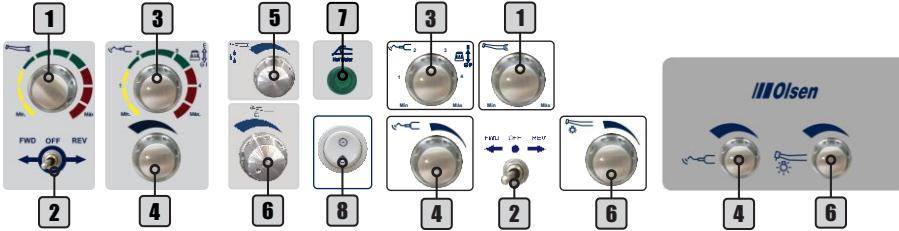
- 1 - Electrical Micromotor Speed Control
- 2 - Electrical Micromotor Direction Control
- 3 - Scaler Power Control and General Function Selector - Perio/Endo
- 4 - Scaler Water Control
- 5 - Brushless Micromotor Water Control
- 6 - Optical Fiber Water Control
- 7 - Thermo Comfort LED On/Off
- 8 - Fiber Optics On/Off

Flex Arm


The Flex Arm features 2 articulations (1) and pneumatic lock that are disabled by touch in the capacitive handle (2), allowing the working table movement.

Control Panel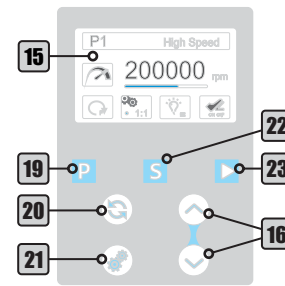

- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down
- 5 - Work Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - On/Off Optical Fiber Light
- 11 - On/Off Air Jet System
- 12 - On/Off Thermo Comfort System
- 13 - X-Ray Viewer

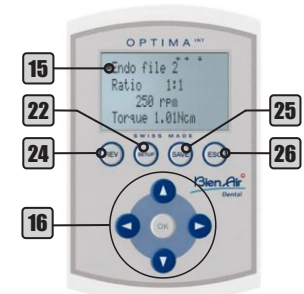- 14 - On/Off X-Ray Viewer
- 15 - Display
- 16 - Adjustment Keys
- 17 - Adjust/Enter Key
- 18 - Reset/Esc Key
- 19 - Program
- 20 - Direction
- 21 - Gear Ratio
- 22 - Setting/Setup
- 23 - Manual Start
- 24 - REV
- 25 - SAVE
- 26 - ESC

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