# OPERATION GUIDE INFINITY

Attention: Carefully read this manual  
before operating the Olsen's equipment!

**Olsen**  
Equipment made to last

## **Index**

1 - Introduction .....	5
2 - Package Contents .....	5
3 - Equipment Presentation .....	5
3.1 - Standard Configuration.....	5
3.1.1 - Dental Chair .....	5
3.1.2 - Assistant Module.....	5
3.1.3 - Operating Light.....	5
3.1.4 - Delivery System.....	5
3.1.5 - Water Unit .....	6
3.2 - Optional Items .....	6
3.3 - Infinity Models .....	6
3.3.1 - Infinity Premium .....	6
3.3.2 - Infinity Cross Flex Premium .....	6
4 - Parts Identification .....	7
5 - Equipment Operating and Description .....	7
5.1 - Before Turning On the Equipment .....	7
5.2 - Turning On the Equipment.....	7
5.2.1 - Delivery System.....	7
5.2.2 - Electric Panel .....	7
5.2.3 - Emergency Stop Switch (Optional) .....	8
5.2.4 - Digital Voltmeter .....	8
5.3 - Remote Foot Pedal Commands.....	8
5.4 - Working Table.....	8
5.4.1 - Infinity Premium Working Table.....	8
5.4.2 - Infinity Cross Flex Premium Working Table.....	10
5.5 - Working Table Arm .....	11
5.6 - Water Unit.....	12
5.7 - Assistant Module.....	12
5.8 - Operating Light.....	12
5.9 - Dental Chair.....	12
5.9.1 - Zero Position .....	13
5.9.2 - Work Position .....	13
5.9.3 - Spit Position .....	13
5.9.4 - Headrest.....	13
5.9.5 - Swivel Armrest.....	14
6 - Instruments Operation .....	14
6.1 - Pneumatic Handpiece Couplings.....	14

6.2 - Optical Fiber Handpiece (Optional).....	14
6.3 - Scaler with LED Light (Optional) .....	14
6.3.1 - Scaler Adjustments - Operation and Basic Adjustments .....	15
6.3.2 - Scaler Adjustments - Multifunction Display .....	15
6.3.3 - Scaler Tips.....	15
6.4 - Electrical Micromotor (Optional) .....	16
6.4.1 - Electrical Micromotor Basic Adjustments .....	16
6.4.2 - Electrical Micromotor Adjustments - Multifunction Display (Optional).....	16
6.4.3 - Brushless Electrical Micromotor - Built in (Optional) .....	16
6.4.4 - Brushless Electrical Micromotor - Side Panel (Optional) .....	17
6.5 - Venturi Saliva Ejector .....	18
6.6 - High Power Vórtice Saliva Ejector.....	18
6.7 - Vacuum Pump Adaptor (Optional) .....	18
6.8 - Way Syringe.....	18
6.9 - Curing Light (Optional) .....	19
6.10 - Prophylaxis System .....	19
6.10.1 - Bicarbonate Jet.....	19
6.10.2 - Prophy-Jet (Optional).....	19
7 - Accessory Operation.....	20
7.1 - ChronoLub Control Panel (Optional).....	20
7.1.1 - ChronoLub System (Optional).....	20
7.1.2 - Clock - Date and Time (Optional).....	20
7.1.3 - Birthday (Optional) .....	20
7.1.4 - Language .....	21
7.1.5 Cardiac Monitor (Optional).....	21
7.2 - Thermo Comfort (Optional) .....	21
7.3 - Air Jet (Optional) .....	21
7.4 - X-Ray Viewer .....	22
7.5 - Anti-Stress System (Optional) .....	22
7.6 - Monitor Holder (Optional) .....	22
7.7 - USB Charger .....	22
7.8 - Cart System (Optional) .....	22
8 - Symbology.....	23
9 - Olsen Accredited Technical Assistance Network.....	24
10 - Warranty Terms.....	25
11 - Message from the President.....	26

# 1 - Introduction

Congratulations on choosing an Olsen equipment!

You have acquired high-tech equipment developed to offer the best performance in dental procedures ensuring ergonomics and comfort, for both professional and the patient.

The purpose of this guide is to present instructions for operating the dental equipment of the Infinity line. The images are merely illustrative and do not represent the configuration of models or the availability of items and accessories.

The *User Manual Dental Line* is supplementary to this guide and should be read before operating the equipment. It is provided in digital format (PDF) along with the order confirmation for your equipment. The user manual presented the descriptive and technical information, installation instructions, maintenance, cleaning, precautions of use and regulatory guidelines.

## 2 - Package Contents

The Infinity line equipment is available in 2 packages, one for the chair and the other for the equipment. Check the contents of each package below:

### Dental Chair Box:

#### Standard Items:

- 1 Dental Chair Infinity
- 1 Accessories Box

#### Optional Items:

- Separated Junction Box

### Delivery Unit Box:

#### Standard Items:

- 1 Delivery Unit Infinity
- 1 Operation Light
- 1 Packing Support
- 1 Dental Stool
- 1 Infinity Quick Operation Guide

#### Optional Items:

- Handpieces
- Cart System
- Additional Stool
- Monitor Holder

**Note:** items availability in the packaging may change in the case of the customer order.

## 3 - Equipment Presentation

### 3.1 - Standard Configuration

#### 3.1.1 - Dental Chair

- 3 Programmable Work Positions
- Automatic Zero Position
- Spit Position
- Multi-Articulating Headrest with “Press-Click” Adjustment System
- Digital Voltmeter
- Swivel Right Armrest
- Integrated Junction Box
- Automatic Vapor Drain

#### 3.1.2 - Assistant Module

- Additional 3-Way Syringe
- Venturi Saliva Ejector
- Bicarbonate Jet Prophylaxis System
- Stainless Steel Tray
- Blue Touch Panel

#### 3.1.3 - Operating Light

- Premium LED 8.000 to 30.000 Lux
- On/Off on the Headlight (Sensor and Knob), on the pedal and on the Command Panel
- Removable Side Handles
- Multifaceted Mirror

#### 3.1.4 - Delivery System

- Couplings for Low-and High-speed Handpieces
- 3-Way Syringe
- Stainless Steel Tray
- Flex Arm with Capacitive Sensor
- Pedal for Chair and Handpieces Commands
- Blue Touch Panel
- USB Charger
- X-Ray Viewer
- Thermo Comfort System

### 3.1.5 - Water Unit

Swivel at 90°  
Translucent Spittoon Bowl  
LED Curing Light  
Bowl Flush Water Control and Cup Filler with Timer

### 3.2 - Optional Items

Leather Upholstery  
Anti-Stress System  
Emergency Stop Button  
Motion Blocking Function  
Anti-crushing System  
Cart System  
Additional Handpiece Position  
Air Jet System  
Optical Fiber Handpiece Coupling  
Electrical Micromotor Brushless  
Electrical Micromotor  
Scaler with LED Light  
Midwest Couplings

ChronoLub System with Multifunction Display  
Cardiac Monitor  
Monitor Holder  
Handpiece Coupling on Assistant Module  
Vacuum Pump Adaptor Set  
High Power Vórtice Saliva Ejector  
Quick Coupler for Saliva Ejectors and/or Vacuum Pump  
Prophy-Jet System  
Additional LED Curing Light on the Assistant Module  
Junction Box  
Prophylaxis in the Assistant Module

**Note:** optional items are defined on the customer's order.

### 3.3 - Infinity Models

#### 3.3.1 - Infinity Premium

It presents the standard items of the Infinity line (section 3.1).

#### 3.3.2 - Infinity Cross Flex Premium

It presents, in addition to the standard items of the Infinity line (section 3.1):

Cross Flex Working Table  
Jointed Stainless Steel Tray  
Coupling for Air Turbines Handpiece

## 4 - Parts Identification

- A - Delivery System
- B - Electrical Panel
- C - Remote Foot Pedal
- D - Working Table
- E - Assistant Module
- F - Operating Light
- G - Dental Chair
- H - Water Unit

## 5 - Equipment Operating and Description

### 5.1 - Before Turning On the Equipment

- 1° - Check if the equipment is properly installed in accordance with the instructions in *chapter 6 (Installation) of the User Manual Dental Line*;
- 2° - Turn On the air compressor that supplies the equipment;
- 3° - Open the main water supply valve of the equipment;
- 4° - Turn On the equipment's power circuit breaker.

### 5.2 - Turning On the Equipment

- 1° - Turn On the On/Off switch (4);
- 2° - Activate the "Seat Up" command on the pedal (section 5.3) until the seat reaches its maximum height limit;
- 3° - Open the protection cover (3);
- 4° - Close the pressure valve (1);
- 5° - Make sure the water tanks (2) are filled with water. If necessary, fill it with drinkable water up to the limit indicated in each tank and reattach them again. If you wish to place prophylactic products (low concentration);
- 6° - Open the pressure valve (1) and check if the water tanks (2) are well attached and free of leaks;
- 7° - Close the protection cover (3);
- 8° - Before starting to use the equipment, check the operation of all available instruments and controls and perform cleaning and sterilization of the instruments (*chapter 7 of the User Manual Dental Line - Cleaning and Disinfection*).

**Do not move the chair with the protection cover open.**

#### 5.2.1 - Delivery System

- 1 - Pressure Valve
- 2 - Water Tanks
- 3 - Protection Cover

#### 5.2.2 - Electric Panel

- 4 - On/Off Switch
- 5 - Fuse Holder
- 6 - Digital Voltmeter
- 7 - Emergency Stop Button (Optional)

### 5.2.3 - Emergency Stop Switch (Optional)

The emergency stop switch interrupts the equipment power supply of and consequently interrupts any running chair movement. To activate the emergency stop switch, simply press it. To turn it off, simply turn it clockwise until the button unlocks, returning to the starting position.

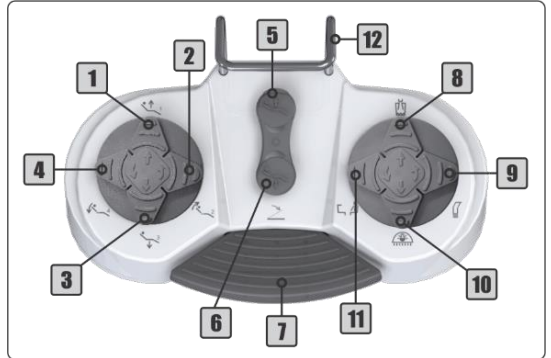
**The emergency stop switch must be deactivated for the equipment operation.**

### 5.2.4 - Digital Voltmeter

When the On/Off switch is turned On, the voltmeter display will show the equipment mains voltage.

## 5.3 - Remote Foot Pedal Commands


- 1 - Seat Up (↑)/Position 1
- 2 - Backrest Up (↖)/Position 2
- 3 - Seat Down (↓)/Position 3
- 4 - Backrest Down (↘)/Spit Position
- 5 - Work Position
- 6 - Zero Position
- 7 - Propulsion Pedal
- 8 - Cup Filler
- 9 - On/Off Vórtice Saliva Ejector
- 10 - On/Off Operating Light
- 11 - Spittoon Bowl Flush
- 12 - Metal Handle

## 5.4 - Working Table

### 5.4.1 - Infinity Premium Working Table

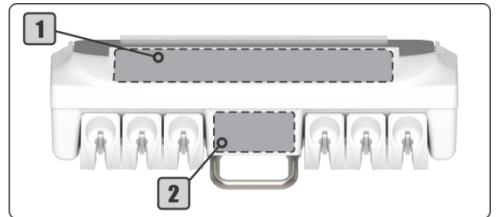- 1 - Command Panel
- 2 - Instruments Control Panel


**The layout of the controls and panels may vary according to customer order and available handpieces.**

**Some functions indicated on control panel may not be available according to the equipment configuration.**

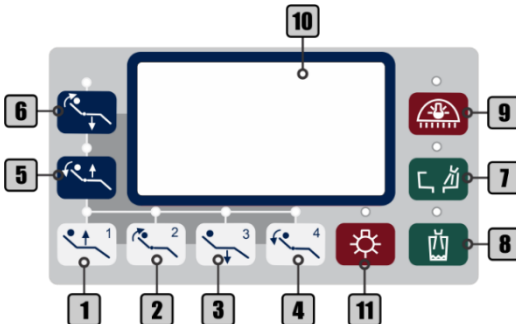
Command Panel Models (1):

- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Spit Position
- 5 - Work Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - X-Ray Viewer
- 11 - On/Off X-Ray Viewer


## ChronoLub System and Multifunctional Display

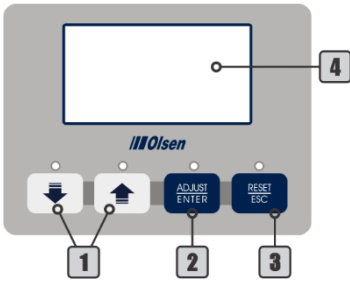
- 1 - Adjustments
- 2 - Adjust/Enter
- 3 - Reset/Esc
- 4 - Multifunctional Display

## Brushless Electrical Micromotor (Optional)

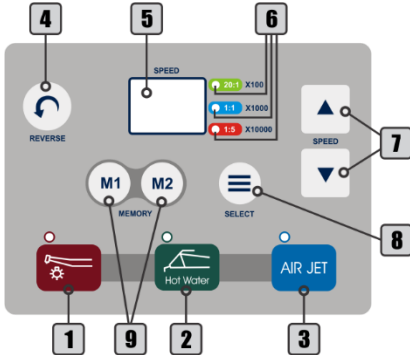- 1 - On/Off Optical Fiber Light
- 2 - On/Off Thermo Comfort
- 3 - On/Off Air Jet
- 4 - Reverse
- 5 - Display
- 6 - Transmission Indicators
- 7 - Adjustments
- 8 - Select
- 9 - Memory

## Brushless - Side Panel (Optional)

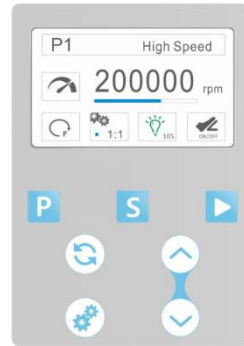Description of controls in section 6.4.4


## Instruments Control Panel (2):

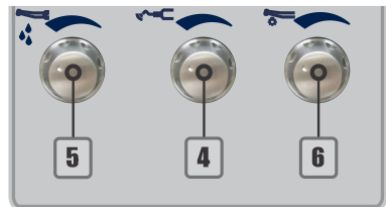The switches and knobs presented below can be presented on the instrument panel according to the instruments available at the dental table.


- 1 - Electrical Micromotor Speed Control
- 2 - Electrical Micromotor Direction Control
- 3 - Scaler Power Control and General Function Selector - Perio/Endo
- 4 - Scaler Water Control
- 5 - Brushless Micromotor Water Control
- 6 - Optical Fiber Water Control

## Version with ChronoLub System

## 5.4.2 - Infinity Cross Flex Premium Working Table

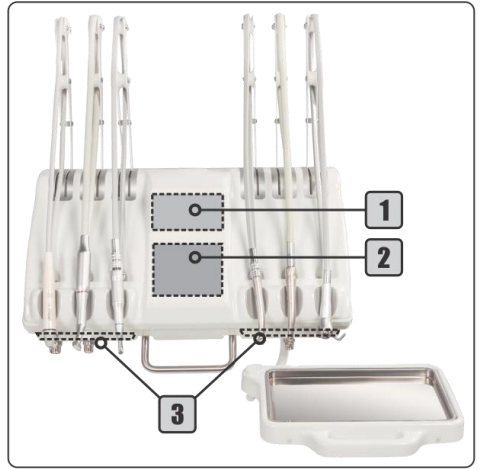- 1 - Top Panel Cross
- 2 - Command Panel
- 3 - Instruments Control Panel


The layout of the controls and panels may vary according to customer order and available handpieces.

Some functions indicated on control panel may not be available according to the equipment configuration.


### Top Panel Cross Flex (1) Models:
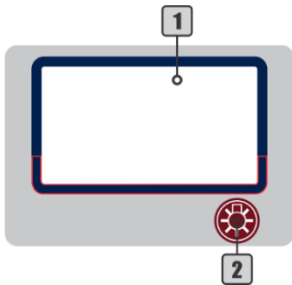
#### X-Ray Viewer


- 1 - X-Ray Viewer
- 2 - On/Off X-Ray Viewer

#### ChronoLub System (Optional)

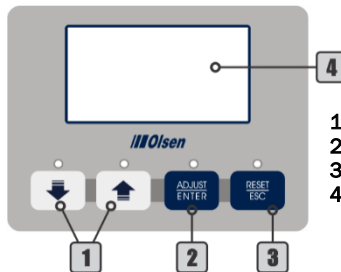
- 1 - Adjustments
- 2 - Adjust/Enter
- 3 - Reset/Esc
- 4 - Multifunctional Display

### Command Panel (2):

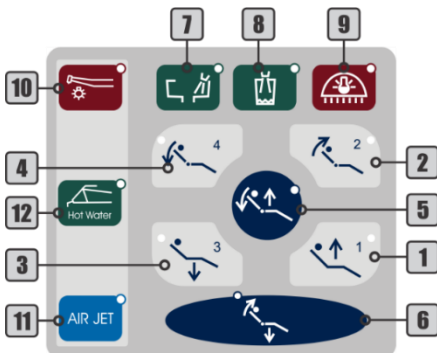
- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Spit Position
- 5 - Work Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - On/Off Optical Fiber Light
- 11 - On/Off Air Jet
- 12 - On/Off Thermo Comfort

### Brushless Electrical Micromotor (Optional)

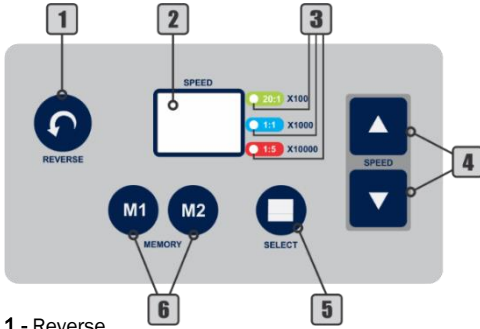
- 1 - Reverse
- 2 - Display
- 3 - Transmission Indicators
- 4 - Adjustment Keys
- 5 - Memory
- 6 - Selection

### Brushless - Side Panel (Optional)

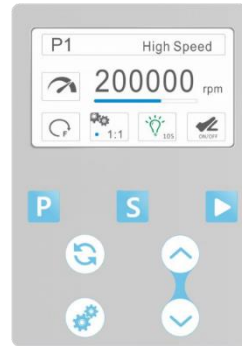Description of controls in section 6.3.3

### Instruments Control Panel (3):

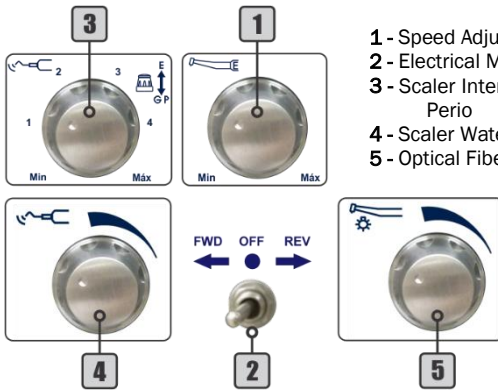- 1 - Speed Adjustment for Electrical Micromotor
- 2 - Electrical Micromotor's Rotation Direction Control
- 3 - Scaler Intensity Control and Function Selector - Endo/General-Perio
- 4 - Scaler Water Control
- 5 - Optical Fiber Water Control


**Note:** these controls are provided below the instrument to which they refer.

## 5.5 - Working Table Arm

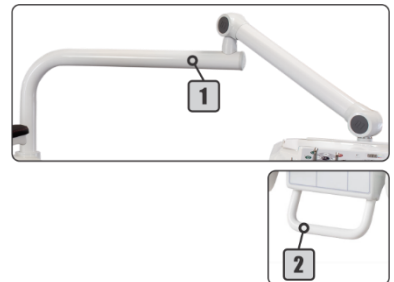**Flex Arm (5 and 6):** features 2 articulations and pneumatic lock that are disabled by touching the capacitive handle (2), allowing the working table movement.

**Do not move the table upright without disabling the pneumatic brake. This may damage the flex arm locking system.**

## 5.6 - Water Unit

**For water commands:** use water commands on remote foot pedal (section 5.3) or command panels on working table (sections 5.4) and assistant module (sections 5.7).

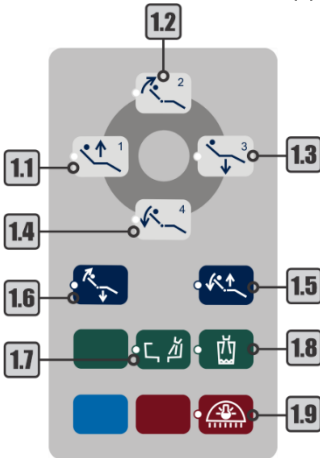
The water spouts for spittoon bowl (1) and cup filler (2) are removable.


## 5.7 - Assistant Module

- 1 – Assistant Module Electrical Panel;
- 2 – Sodium Bicarbonate Reservoir;
- 3 – Stainless-Steel Tray.

Assistant Module Commands (1):

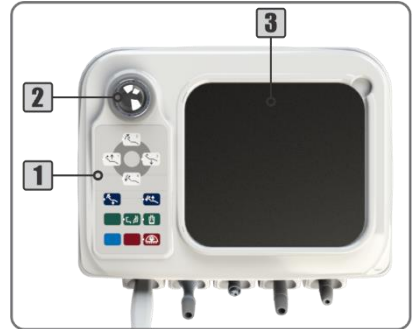- 1.1 - Seat Up/Position 1
- 1.2 - Backrest Up/Position 2
- 1.3 - Seat Down/Position 3
- 1.4 - Backrest Down/Spit Position
- 1.5 - Working Position
- 1.6 - Zero Position
- 1.7 - Spittoon Bowl Flush
- 1.8 - Cup Filler
- 1.9 - On/Off Operating Light

**Note:** the layout of the assistant module may vary according to customer order and available handpieces.

## 5.8 - Operating Light

**To activate the operating light:** press the On/Off operating light command (1) on the command panel (section 5.4 and 5.7) or remote foot pedal (section 5.3) to enable sensor (3) and control knob (2).

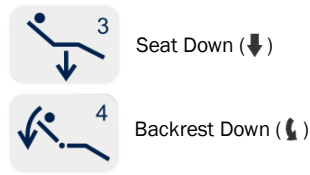
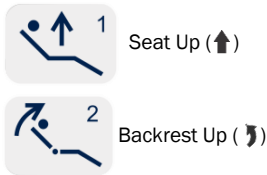
**To turn On/Off the operating light:** pass the hand in front of the sensor (3) approximately 5 cm away or press the control knob (2).

**To change the light intensity:** hold your hand in front of the sensor (3) or turn the control knob (2) until the operating light shows the desired intensity.

## 5.9 - Dental Chair

To move backrest and seat use the keys on the command panel (sections 5.4 and 5.7) and in the buttons located on remote foot pedal (section 5.3).

### 5.9.1 - Zero Position

Press the Zero Position key to automatically adjust the seat to the lowest position and backrest in the highest position simultaneously, to facilitate patient entry and exit.

### 5.9.2 - Work Position

**To save a work position:**

- 1° - Press the Zero Position key (*section 5.8.1*);
- 2° - Adjust the backrest to the desired position;
- 3° - Adjust the seat to the desired position;
- 4° - Press the Work Position key for 5 seconds. The equipment will emit 2 long beeps;
- 5° - Press command 1 (Seat Up) while the equipment beeps; The equipment will emit 2 short beeps to confirm the procedure;

**Note:** the time to record the working position is only 2 seconds between *steps 4 and 5*, then when you hear the first long beep (*step 4*), quickly press one of the available commands (1 to 3) to record the working position. If none of the 3 commands is pressed in these 2 seconds, the equipment will emit 3 short beeps, canceling the operation.

- 6° - Press the Zero Position key.

Repeat the previous steps by replacing the command 1 by the command 2 (Backrest Up) to record the working position 2 or by the command 3 (Seat Down) to save the working position 3.

**To perform a working position:**

- 1° - Press the Work Position key; The equipment will emit a short beep;
- 2° - Press the command for the desired saved working position (1 to 3); The equipment beeps twice confirming the operation.

The dental chair moves the backrest and seat simultaneously to the previously saved position and lights the operating light at the last intensity used.

**Note:** after pressing the Work Position key, the time for position selection is 4 seconds. If no key is pressed within this time, the equipment cancels the operation.

### 5.9.3 - Spit Position

**To perform the spit position:**

- 1° - Press the Work Position key (*section 5.8.2*);
- 2° - Press the command 4 (Backrest Down).

The equipment emits 2 beeps, turn Off the operating light, the backrest goes up and activates the water in the bowl. After 8 seconds, the equipment returns to the position before the command.

**Note:** the spit position is pre-recorded at the factory and cannot be changed.

### 5.9.4 - Headrest

**To adjust the headrest (2) position:** press the button (1) to move the headrest (2) to the desired position, then release the button.

**To adjust the height of the headrest:** move the bar (3) until you reach the position. The headrest bar (3) allows adjustment with up to 10 cm from the backrest.

### 5.9.5 - Swivel Armrest

To move the swivel armrest: press the trigger (1) and push the armrest down.

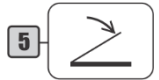
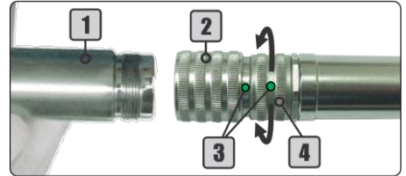
## 6 - Instruments Operation

### 6.1 - Pneumatic Handpiece Couplings

Before starting the operation connect the handpiece (1) to the coupling by attaching it to the coupling ring (2), ensuring that it is properly connected and that there is no leakage of air or water.

To activate the rotation: press the propulsion/activation pedal (5). The higher the pressure applied to the pedal, the more speed in the handpiece, until reaching the pressure limits of the equipment.

To adjust the spray: turn the adjusting ring (4) until the desired volume of water. To adjust the maximum flow of water, bring the spray adjustment indicators (3) closer to each other. To decrease the flow of water, rotate the ring so it distances one indicator from the other. The completely opposite points indicate the minimum water regulation for the spray.



### 6.2 - Optical Fiber Handpiece (Optional)

Before starting the operation connect the optical fiber handpiece to the coupling by attaching it to the coupling ring (1), ensuring that it is properly connected and there is no leakage of air or water.

To activate the rotation: press the propulsion/activation pedal (2). The higher the pressure applied to the pedal, the more speed in the handpiece, until reaching the pressure limits of the equipment.

To turn the coupling's light On/Off: use the On/Off optical fiber light (3) on the command panel (section 5.4 and 5,7) of the working table.

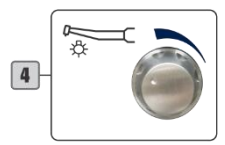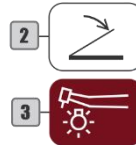To adjust the spray: use the optical fiber water control knob (4) on the instrument control panel.



### 6.3 - Scaler with LED Light (Optional)

Before starting the scaler operation, install the tip on the transducer (1) and connect the transducer (1) to the coupling (3) checking if are connected.

To attach the transducer to the coupling: align the adjustment indicators (2) of the two parts and carefully insert them.

To install the scaler tips: insert the tip into the transducer (1) by threading it carefully, then insert the Torque wrench (4) into the tip and then rotate it clockwise until it is tight. To remove the tip, turn it counterclockwise with the Torque wrench (4).

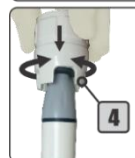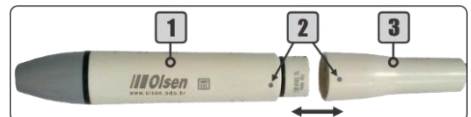To install a file adapter (not supplied with scaler): attach the adapter to the transducer (1) and screw it

gently. Insert the file into the tip of the adapter, thread the tip and then use the Endo wrench (5) to tighten. To remove the adapter, use the Endo wrench (5) to loosen it and then carefully unscrew.

### 6.3.1 - Scaler Adjustments - Operation and Basic Adjustments

**To activate the scaler:** push the propulsion/activation pedal (6).

**To adjust the vibration power:** turn the control knob (7) until the desired power is reached.

**To toggle functions:** push the control knob (7) in direction of the table for "GP" function (General and Perio). Pull the knob gently in the opposite direction of the table for "E" (Endo) function.

**To adjust the water (including for multifunction display):** use the control knob (8) on the instrument control panel (section 5.4).

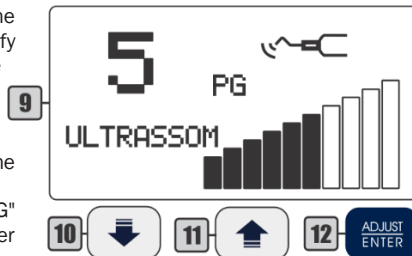
### 6.3.2 - Scaler Adjustments - Multifunction Display

When removing the scaler from the holder or extending the rod (Cross Flex), the multifunctional display (9) screen will identify the scaler, the number of its position on the working table (1 to 5), the selected function ("PG" or "E") and will display the vibration power bar.

**To adjust the vibration power:** use the keys to increase (11) or decrease (10) the power, which will be indicated on the display.

**To toggle functions:** use the key (12) to toggle between "PG" (General and Perio) and "E" (Endo) functions indicated in the center of the multifunctional display.

**! It is not possible to switch the PG/E functions during the operation of the scaler (while propulsion/activation pedal is activated).**

### 6.3.3 - Scaler Tips

The tips GD1, PD1 and PD3 accompany the scaler. The power and application information for each tip is available in the table below.

Tip	Code	Function	Power	Indication of use
	GD1	G	1-10	Removal of calculus and plates in the supragingival, interdental and tooth surfaces.
	PD1	P	1-10	Removal of calculus subgingival
	PD3	P	1-6	Removal of deep calculus subgingival

**Note:**

- 1 - Do not use the tip on features other than those specified by the manufacturer.
- 2 - Do not exceed the power limit for each tip. That could damage the tip and the scaler.

## 6.4 - Electrical Micromotor (Optional)

Before starting the operation connect the handpiece to the electrical micromotor (2).

**To activate the rotation:** push the propulsion/activation pedal (1).

### 6.4.1 - Electrical Micromotor Basic Adjustments

**To adjust the speed:** turn the control knob (3) until the desired speed is reached.

**To change the direction of rotation:** use the rotation direction control (4).

- **FWD:** clockwise;      - **OFF:** Off;      - **REV:** counterclockwise.

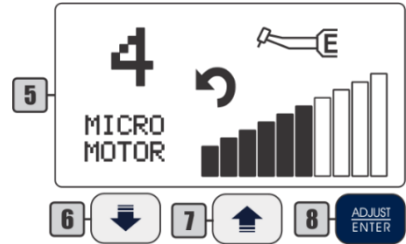
### 6.4.2 - Electrical Micromotor Adjustments - Multifunction Display (Optional)

When removing the electrical micromotor from the holder or extending the rod (Cross Flex), the multifunction display (5) will identify the electrical micromotor, the number of its position on the working table (1 to 5), the direction of rotation and will show the speed bar.


**To adjust speed:** use keys to increase (7) or decrease (6) speed.

**To change the direction of rotation:** use the Adjust/Enter key (8) to toggle the direction of rotation.

↻ (FWD) Clockwise      ↻ (REV) Counterclockwise

### 6.4.3 - Brushless Electrical Micromotor - Built in (Optional)

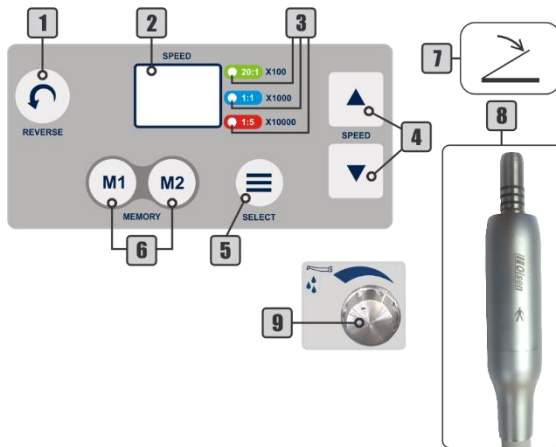Valor no Display	Velocidade (RPM)		
	20:1 (x100)	1:1 (x1.000)	1:5 (x10.000)
01	100	2,000	10.000
02	200	4,000	20.000
03	300	6,000	30.000
04	400	8,000	40.000
05	500	10,000	50.000
06	600	12,000	60.000
07	700	14,000	70.000
08	800	16,000	80.000
09	900	18,000	90.000
10	1,000	20,000	100.000
11	1100	22,000	110.000
12	1200	24,000	120.000
13	1300	26,000	130.000
14	1400	28,000	140.000
15	1,500	30,000	150.000
16	1600	32,000	160.000
17	1700	34,000	170.000
18	1800	36,000	180.000
19	1900	38,000	190.000
20	2,000	40,000	200.000
21	2100	42,000	210.000
22	2200	44,000	220.000
23	2300	46,000	230.000
24	2400	48,000	240.000
25	2,500	50,000	250.000
26	2600	52,000	260.000
27	2700	54,000	270.000
28	2800	56,000	280.000
29	2900	58,000	290.000
30	3,000	60,000	300.000
31	3100	62,000	310.000
32	3200	64,000	320.000
33	3300	66,000	330.000
34	3400	68,000	340.000
35	3,500	70,000	350.000
36	3600	72,000	360.000
37	3700	74,000	370.000
38	3800	76,000	380.000
39	3900	78,000	390.000
40	4,000	80,000	400.000

Before starting the operation, connect the handpiece to the electrical micromotor (8).

**To activate the rotation:** press the propulsion/activation pedal (7).

**To select the transmission:** use the Select key (5). The selection is indicated by LED's (3): 20: 1; 1: 1 and 1: 5.

**To reverse the rotation:** use the Reverse key (1).

**To adjust speed:** use the adjustment key (4). The speed relation of each transmission with the value of the display (2) is indicated in the above table.

Example:

Display value: 15

Selected gear ratio: 1:1 Factor: x1.000

Rotation speed: 15 x 1000 = 15,000 rpm

**To activate memory:** press the desired Memory key (6). The display will show the speed and the LED (3) will indicate the present transmission.

**To record memory:** select the transmission and adjust the speed of rotation to store. Press and hold one of the Memory key (7) until the system emits 2 beeps, indicating that the memory has been recorded.

To save the second available memory repeat the same procedure.

**To adjust the electrical micromotor water:** use the water control knob (9) on the instruments control panel.

### 6.4.4 - Brushless Electrical Micromotor - Side Panel (Optional)

Before starting the operation, connect the handpiece to the electrical micromotor (16).

**To activate the rotation:** press the propulsion/activation pedal (14) or the Manual Start key (11). The On/Off indicator (2) will turn blue.

**To select a preset program:** use the Program key (8). There is 7 programs available, according the table below:

Program	Default settings
P1 My favorite1	
P2 My favorite2	
P3 Tooth prep (H)	1:5 200.000 rpm
P4 Tooth prep (L)	1:5 20.000 rpm
P5 Contra angle	1:1 40.000 rpm
P6 Straight	1:1 40.000 rpm
P7 Polishing	1:1 5.000 rpm

The selected direction is shown on the display (1).

**To record program:** select program P1 or P2, then select the desired gear ration and adjust the speed. Press and hold the Program key (8) until the system save your settings.

**To adjust speed:** use the adjustment keys (13) to increase or decrease speed. Please see the table below for speed range for each gear ration value:

Gear ratio	Speed range (rpm)
1:5	10.000 - 200.000
1:1	2.000 - 40.000
4:1	500 - 10.000
10:1	200 - 4.000
16:1	100 - 2500
20:1	100 - 2000

The selected direction is shown on the display (5).

**To select the gear ratio:** use Gear Ratio key (10). There are the following values available: 1:5, 1:1, 4:1, 10:1, 16:1 and 20:1.

The selected direction is shown on the display (4).

**To change the rotation direction:** use the Direction key (9).

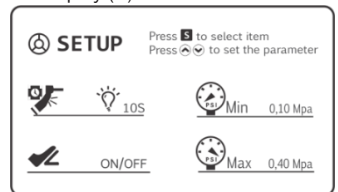
The selected direction is shown on the display (3).

**To adjust the micromotor water:** use the water control knob (15) on the instruments control panel (section 5.4).

**To access setup menu:** press and hold the Setting key (12). Use the Setting key (12) to select the desired option.


**To adjust LED options:** use the adjustment keys (13) to set the parameter.

LED Off	LED On without delay	LED On with 10s delay



**To adjust pneumatic activation method:** use the adjustment keys (13) to set the parameter. The selected pneumatic activation method is shown on the display (6).

Micromotor performs speed according to the pressure on the pedal (14)	When pedal (14) is pressed micromotor performs the maximum speed adjusted

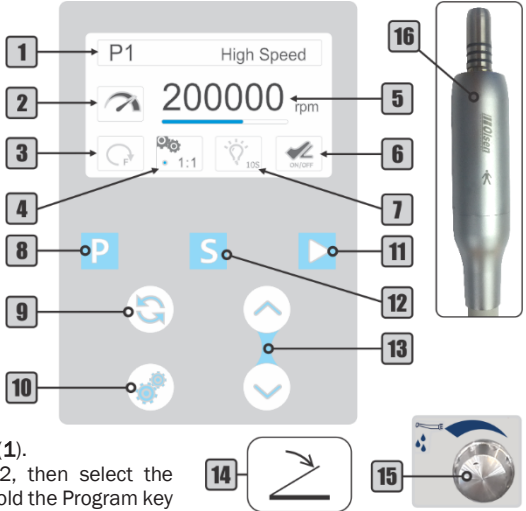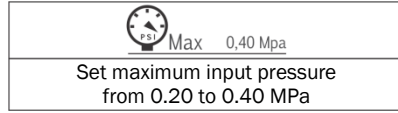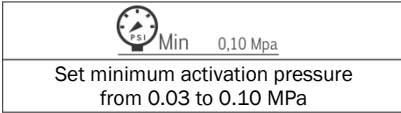**To adjust input pneumatic pressure:** use the adjustment keys (13) to set the parameter.

**Note:** the micromotor system preset by the factory according to the internal equipment pressure. It is not recommended to change the pressure parameters.

## 6.5 - Venturi Saliva Ejector

Before starting the operation, fit the cannula to the  $\varnothing 6,3$  or  $9,35$  mm adapter (3).

**To activate the suction:** remove the ejector (2) from the holder (1).

**To deactivate the suction:** place the ejector in its holder (1).

The cannula adapter (3) is removable for cleaning.

## 6.6 - High Power Vórtice Saliva Ejector

Before starting the operation, fit the cannula to the  $\varnothing 6,3$ ,  $\varnothing 9,5$  or  $\varnothing 11$  mm adapter (3).

**To enable/disable suction for Vórtice ejector (2):** use the On/Off Vórtice saliva ejector button (1) on the pedal.

The cannula adapter (3) is removable for cleaning.

## 6.7 - Vacuum Pump Adaptor (Optional)

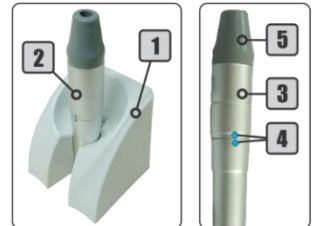
Before starting the operation, fit the cannula to the  $\varnothing 6,3$ ,  $\varnothing 9,5$  or  $\varnothing 11$  mm adapter (5).

**To activate suction:** remove the vacuum pump suction (2) from its holder (1). The vacuum pump will work automatically.

**To adjust suction flow:** to decrease the suction flow, turn adaptor's body (3) away from the indicators (4). The more distant, the smaller the suction flow. For the maximum suction flow, align the indicators (4).

**To deactivate the suction:** place the vacuum pump in its holder (1).

The cannula adapter (5) is removable for cleaning.


## 6.8 - Way Syringe

Before beginning the use of the 3-way syringe (5), connect the syringe tip (3) by pressing the locking ring (4) into the correct fit.

**To emit air jet:** press the air button (1).

**To emit water jet:** press the water button (2).

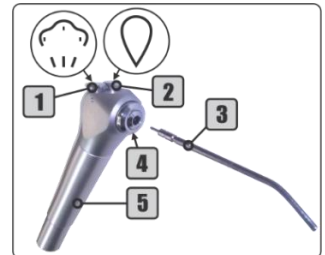**To emit spray jet:** press simultaneously the air (1) and water (2) buttons.

## 6.9 - Curing Light (Optional)

Before starting the curing light use, install the tip (1) and the light hood (2).

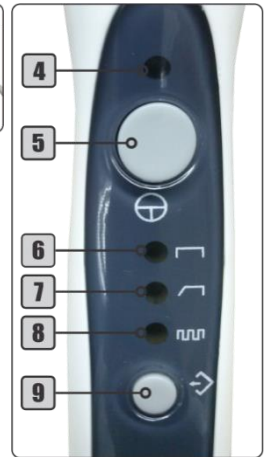
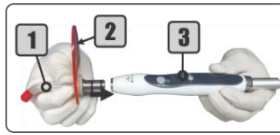
**To install the tip:** first, attach the light hood (2) to the tip (1) and then attach it to the curing light unit (3). The tip (1) must be pushed all the way to the fitting end.

**To activate/deactivate the curing light:** press the On/Off button (5) twice. The green LED (4) indicates that the curing light is operating. The curing light unit will run for 20 seconds and will turn Off automatically.

**To toggle the operating mode:** each tap on the selection button (9) switches the operation mode. Press the selection button (9) until the orange LED indicates the desired mode.

**The curing light operates in the following modes:**

- Continuous (6): turns on the maximum power for 20 seconds;
- Ramp (7): turns on the minimum power, gradually increasing. In 5 seconds reaches its maximum power until the cycle of 20 seconds is finished;
- Pulse (8): the device blinking for 20 seconds.

## 6.10 - Prophylaxis System

### 6.10.1 - Bicarbonate Jet

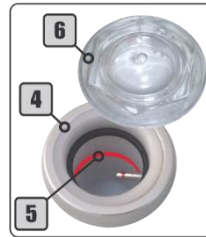
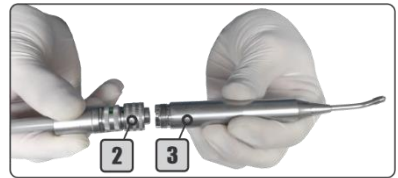
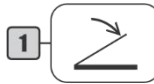
Before starting use of the bicarbonate jet, its necessary deposit bicarbonate sodium in the reservoir (4), check for moisture in the filter (8) and connect the bicarbonate jet handpiece (3) to its coupling (2).

**To fill the bicarbonate reservoir:** open the reservoir (4) by turning the cover (6) anticlockwise and deposit the bicarbonate sodium until to the red line (5). Close the reservoir.

**To remove condensed water in the filter:** press the valve (7) on the base of the filter (8) to drain the water.

**To activate/deactivate the bicarbonate jet:** press the progressive/activation pedal (1). The more pressure on the pedal, the greater the volume of the jet emitted by the handpiece, until reaching the pressure limits of the equipment.

To adjust the spray: see section 6.1.

**Clean the sodium bicarbonate tank (4) at the end of each procedure.**

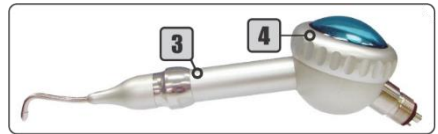
### 6.10.2 - Prophy-Jet (Optional)

Before starting Prophy-Jet (3), deposit up to 15g of sodium bicarbonate for dental use in the sodium bicarbonate tank (4) and attach the Prophy-Jet to the pneumatic coupling (2) available on the assistant module (section 5.7) or pneumatic coupling for low-speed rotation in the working table (section 5.4).

**To activate/deactivate:** push the propulsion/activation pedal (1). The higher the pressure applied to the pedal, the greater the volume of the jet emitted by the handpiece, until reaching the pressure limits of the equipment.

**Clean the sodium bicarbonate tank (4) at the end of each procedure.**

## 7 - Accessory Operation

### 7.1 – ChronoLub Control Panel (Optional)

- 1 - Decreases
- 2 - Increases
- 3 - Adjust/Enter
- 4 - Reset/Esc

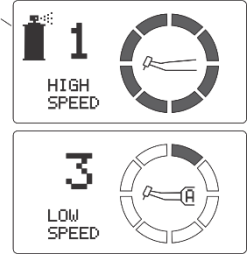
#### 7.1.1 - ChronoLub System (Optional)

When removing the handpiece from the holder or extending the rod (Cross Flex), the multifunction display will identify whether the coupling is for low or high rotation, show the number of its position on the working table (1 to 5), and a segmented circle that represents the time of use of the handpiece. When the last segment starts blinking, the handpiece can be used for another 2 minutes.

When you run out of time, the screen will show a spray can and will emit 3 beeps. From this moment whenever the handpiece is removed from the holder, the system will beep, repeating the warning every 2 minutes while the handpiece is in use or out of the holder.

After lubricating the handpiece, with it out of the holder or extending the rod (Cross Flex), press the Reset/Esc key (4) for 3 seconds to restart the timing. This procedure can be done at any time.

#### 7.1.2 - Clock - Date and Time (Optional)

**To access the Clock function:** press the Adjust/Enter key (3) for 3 seconds to access the "AJUSTE" screen. Use the adjustment keys (1 and 2) to select the "1. RELÓGIO" option.

**To adjust the clock:** press the Adjust/Enter key (3) to access the "AJUSTAR RELÓGIO" screen. Use the adjustment keys (1 and 2) to change the selected value.

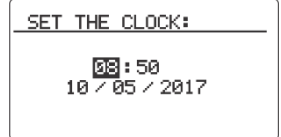
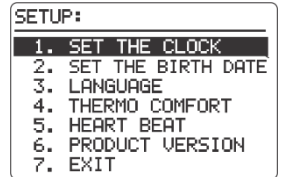
**To toggle the selection between hour, minute, day, month and year:** use the Adjust/Enter key (3).

**To save the changes:** press the Adjust/Enter key (3) for 3 seconds.

**To cancel and return to the previous screen:** press the Reset/Esc key (4).

**The options in the "AJUSTE" screen may change if the cardiac monitor (optional) is available on the device. (section 6.11.4).**

#### 7.1.3 - Birthday (Optional)

**To access the Anniversary function:** press the Adjust/Enter key (3) for 3 seconds to access the "AJUSTE" screen. Use the adjustment keys (1 and 2) to select the "2. ANIVERSÁRIO" option.

**To change the anniversary date:** press the Adjust/Enter key (3) to access the "AJUSTE ANIVERSÁRIO" screen. Use the adjustment keys (1 and 2) to change the selected value.

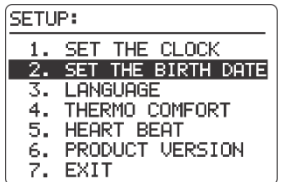
**To change the selection between day, month and year:** use the Adjust/Enter key (3).

**To save the changes:** press the Adjust/Enter key (3) for 3 seconds.

**To cancel and return to the previous screen:** press the Reset/Esc key (4).

**The options in the "AJUSTE" screen may change if the cardiac monitor (optional) is available on the device. (section 6.11.4).**

## 7.1.4 –Language

**To change the language:** press *Adjust/Enter* (3) for 3 seconds to access the "SETUP" screen. Use the arrows (1 e 2) to select the option 3. LANGUAGE.

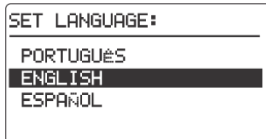
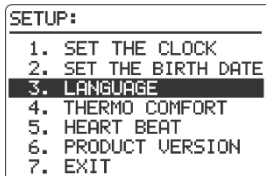
**To select the language:** press *Adjust/Enter* (3) to access "SET LANGUAGE" screen. Use the arrows (1 e 2) to select the wish language.

**To save the alteration:** press *Adjust/Enter* (3) for 3 seconds.

**To cancel and return:** press *Reset/Esc* (4).

**The options in the "AJUSTE" screen may change if the cardiac monitor (optional) is available on the device. (section 6.11.4).**

## 7.1.5 Cardiac Monitor (Optional)

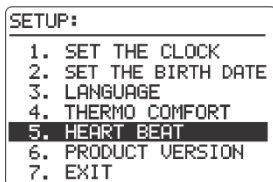
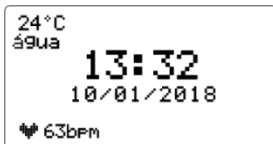
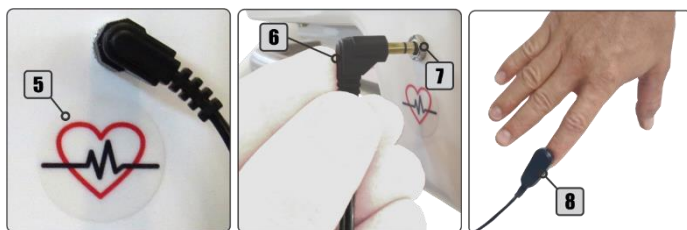
**To activate the cardiac monitor:** identify the input connection (7) identified with the icon (5) and connect the plug (6). Attach the sensor (8) to the tip of any of the patient's fingers.

The patient's cardiac monitoring will automatically appear in the lower left corner of the display after 8 seconds.

**To shown only the cardiac monitor on the display:** press the *Adjust/Enter* key (3) for 3 seconds to access the "AJUSTE" screen. Use the adjustment keys (1 and 2) to select option "4. BATIMENTO".

**To access the "BEAT" screen:** press the *Adjust/Enter* key (3).

**To return to the previous screen:** press the *Reset/Esc* key (4).

## 7.2 - Thermo Comfort (Optional)

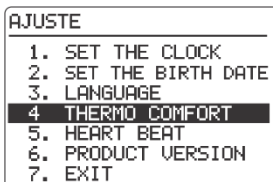
**To activate/deactivate the Thermo Comfort system:** use the Thermo Comfort On/Off key (1) on the worktable.

**To access the Thermo Comfort screen:** press the *Adjust/Enter* key (3) for 3 seconds to access the "AJUSTE" screen. Use the adjustment keys (1 and 2) to select the "3. THERMO COMFORT" option.

**To adjust the water temperature:** press the *Adjust/Enter* key (3) to access the "AJUSTE TEMPERATURA" screen. Use the adjustment keys (1 and 2) to change the temperature. Press the *Adjust/Enter* key (3) to save.


**To cancel and return to the previous screen:** press the *Reset/Esc* key (4).

## 7.3 - Air Jet (Optional)

**To activate/deactivate the Air Jet system:** use the On/Off Air Jet key (1) on the command panel. When Air Jet system is activated, the continuous air jet starts as soon as the handpiece is removed from the holder.

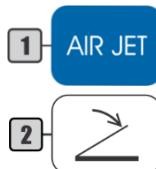When pressing the propulsion/activation pedal (2), the system is interrupted, and the handpiece operates normally. When releasing the propulsion/activation pedal (2), the air jet will restart until the system is turned Off or the handpiece is placed in its holder.

## 7.4 - X-Ray Viewer

To turn On the x-ray viewer: press the On/Off key (1) on the command panel.

## 7.5 - Anti-Stress System (Optional)

To activate/deactivate Anti-Stress system: press the “On/Off” key (5).

To enable the automatic massage system: press the “AUT” key (3). The system will start executing 15 types of massage combinations, alternating one combination every minute.

To toggle massage combinations: use the “AJUSTES” keys (8) while performing automatic massage to advance or repeat the desired massage combination.

To adjust the massage time: press the “TEMPO” key (6) and then use the “AJUSTES” key (8) to adjust the duration of the massage (from 5 to 45 minutes).

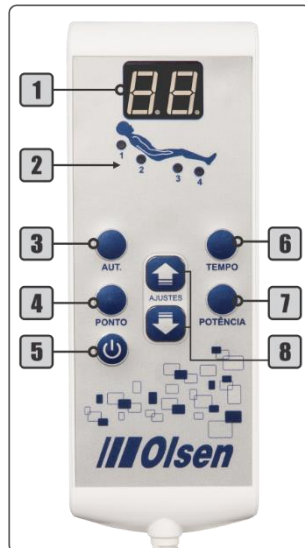
To adjust the vibration power: press the “POTÊNCIA” key (7) and then use the “AJUSTES” key (8) to adjust up to 10 levels of vibration intensity for the massager’s motors.

To enable/disable massagers: use the “PONTO” key (4) to define which massagers should work during the massage. Use the “AJUSTES” key (8) just after the “PONTO” key (4) to customize the running massages. The LED’s (2) indicated the motor that will be active.

To restore factory settings: press the “AUT” key (3) twice quickly.

After 20 seconds after the last command executed, the display (1) will show the initials AU, but without affecting the settings made in the Anti-Stress system.


## 7.6 - Monitor Holder (Optional)

The monitor holder (2) features a VESA plug-in standard (1) to support 14” to 19” monitors. The monitor holder fits perfectly on the dental reflector arm with fixing ring (4) secured with a 3 mm Allen screw (3).

The monitor holder features a rotation on the axis, tilt adjustment and lateral rotation for the monitor.

Do not use on the monitor holder:

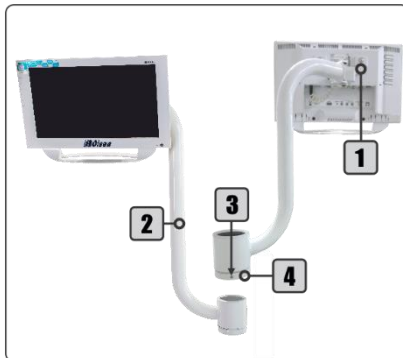- 1 - Equipment powered with more than 24 Volts.
- 2 - Equipment with weight more than 1 kg.

The monitor is not provided with the holder.

## 7.7 - USB Charger

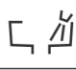
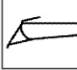
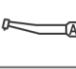
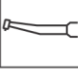
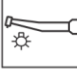
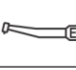
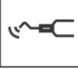
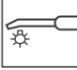
See the *User Manual Dental Line*.

## 7.8 - Cart System (Optional)

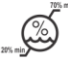
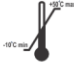
See the *User Manual Dental Line*.

## 8 - Symbology

The symbology complies with IEC 60601-1, IEC 60878, ISO 15223-1, ISO 7010.

	Patient Chair Up (Dental)		Patient Chair Down (Dental)		Zero Position
	Backrest Up		Backrest Back		Work Position
	Bowl Flush		Cup Filler		Multifunction Syringe (Air-Water)
	Dental Air Motor		Turbine		Turbine with Illumination
	Dental Low Voltage Electric Motor		Ultrasonic Scaler		Fibre-Optic Handpiece
	Saliva Ejector		Suction Handpiece		Suction Handpiece with Hand Control Valve
	Variability, for Rotating Movement		Dental Operating Light		Lighting
	Non-Sterile		Hand Control Valve		Foot Switch
	Spray-cooling		"OFF" (Power)		"ON" (Part of the Equipment)
	Stop		"ON" (Power)		"OFF" (Part of the Equipment)
	Emergency Stop		"ON"/"OFF"		Write Data into Store (Selection)
	Manufacturer		Caution		Warning, Electricity
	General Warning Sign		Level		Sterilizable Up to the Temperature Specified
	Operating Instructions		General Prohibition Sign		Refer to the Instruction Manual
	General Mandatory Action Sign		Stepping Prohibited		Type BF Applied Part

	Type B Applied Part		Serial Number		Protective Earth (Ground)
	Earth (Ground)		Keep Away from Heat		Humidity Limitation
	Fragile, Handle with Care		Temperature Limitation		Keep Dry
	Heaping Up		This Side Up		Alternating Current
		Authorized Representative in the European Community			

## 9 - Olsen Accredited Technical Assistance Network

To access the Olsen certified technical assistance network for installation and maintenance, contact us by e-mail [export3@olsen.odo.br](mailto:export3@olsen.odo.br), or by phone +55 48 2106 6000.

 **The installation and maintenance of the equipment and its accessories must only be carried out by Olsen Technicians.**

 **Use Olsen genuine parts only. The use of non-genuine parts, accessories or components may compromise the proper functioning of the equipment.**

 **Do not make adaptations or changes to the equipment, its components or accessories.**

## **10 - Warranty Terms**

The warranty period is of 12 months starting from the equipment's installation date since the installation be made during the 90 days after the purchasing date and under the terms of this certificate.

- 1** - The maximum storage period must be 3 months from the purchasing date. In case of this period is exceeded, the warranty period will start counting even if the product is kept in storage.
- 2** - Upholstered parts are under warranty for 6 months.
- 3** - Operating light mirror, fuses, wires, and transformers are not covered by this warranty.
- 4** - The warranty is limited to the repair or replacement of defected parts and does not cover defects originated by:
  - a) Non-observance of the use and maintenance instructions;
  - b) Falls, crashes and inadequate storage;
  - c) Effects caused by the agents of nature;
  - d) Damage caused to the upholstered parts, due to improper use of chemical products, exposure to inadequate weather conditions, contact with fabrics, leather, disposable gloves, paints, pigmented detergents, cutting or piercing objects;
  - e) Damage to painted parts and plastic covers caused by improper use of chemical products or by contact with disposable gloves and cutting or piercing objects;
  - f) Connection with a wrong voltage power supply.
- 5** - This warranty will cease when:
  - a) The warranty period expires;
  - b) The user makes any changes to the product or uses inappropriate accessories;
  - c) The warranty certificate presents any scraping or changes;
  - d) Installation and technical service is done by an unauthorized technician;
  - e) The equipment is stored for more than 6 months after the invoice issuing date;
  - f) For the use of spare parts other than Olsen originals.
- 6** - The repairing or replacement of parts under the warranty period will not extend its original expiring date.
- 7** - The expenses resulted from the equipment's installation, scheduled maintenance, travel and accommodation costs of the service staff involved in the calls for service for the equipment's installation or repair will run under the owner's responsibility and in accordance with the distributor's norms.
- 8** - The customer, after checking the installation, must keep the purchasing invoice and the equipment's serial number until the warranty period end, as in the case of service needed, the serial number, the invoice number, and purchasing or installation date will be required.
- 9** - On all calls for service during the warranty period the equipment's serial number and a copy of the purchasing or installation invoice must be shown. In the case of this information is not presented, the service will not be done under warranty.

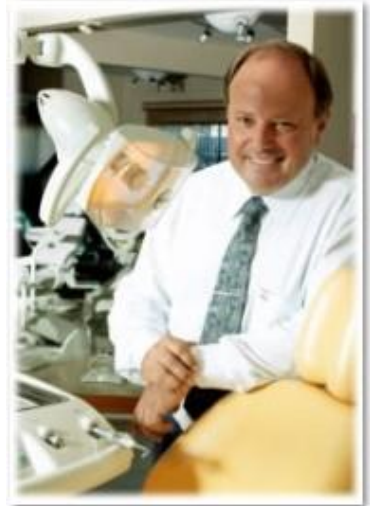
## **11 - Message from the President**

**Olsen and its clients:  
A successful relationship.**

I have linked my name to the factory and to the Dental and Medical equipment that are currently produced and trade in more than 100 countries having in mind the responsibilities and long-term response to this initiative.

Our products are modern, innovative, durable and of low maintenance cost. These characteristics have been achieved thanks to our competent and dedicated team, which make me very proud for many reasons, as they are always giving the best of their creative capacity to our clients.

Our company will always be open to all those who prefer Olsen products, for any necessary information and technical assistance, but especially for comments regarding the relationship with customers. We expect this connection always brings you satisfaction, resulting in more and more benefits to all of us.

Cesar Olsen

[www.olsen.odo.br](http://www.olsen.odo.br) | [loja.olsen.odo.br](http://loja.olsen.odo.br)

+55 48 2106 6000  
[export3@olsen.odo.br](mailto:export3@olsen.odo.br)

# **Olsen**

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Registration at Ministry of Health 1028130009

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