

## Upholstery Replacement

Instruction for upholstery replacement, for seat and legrest upholsteries.

1




Set the seat to higher position and inclinate it. Rise up the legrest and move down the backrest. Equipment should be on the same position of the picture.

2




Find the 6 plastic rivets which hold the legrest plastic cover and remove them. Push the center pin (of plastic rivet) by using a metal rivet or a 1mm Hex screw. After pull it out.

3

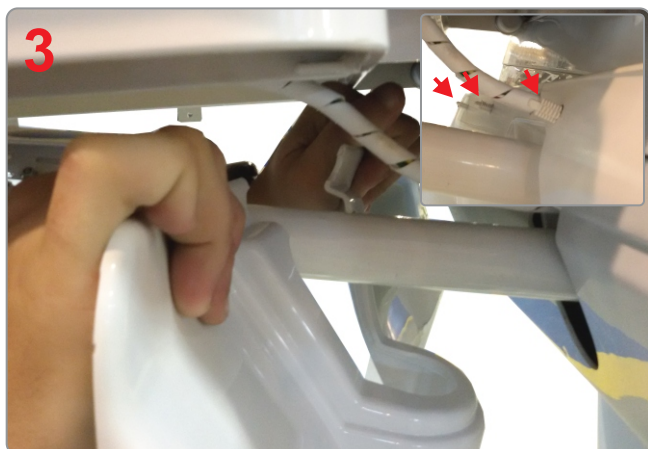


Bend the plastic cover as the image above and remove it from legrest. Please don't remove the three plastic clips.

4

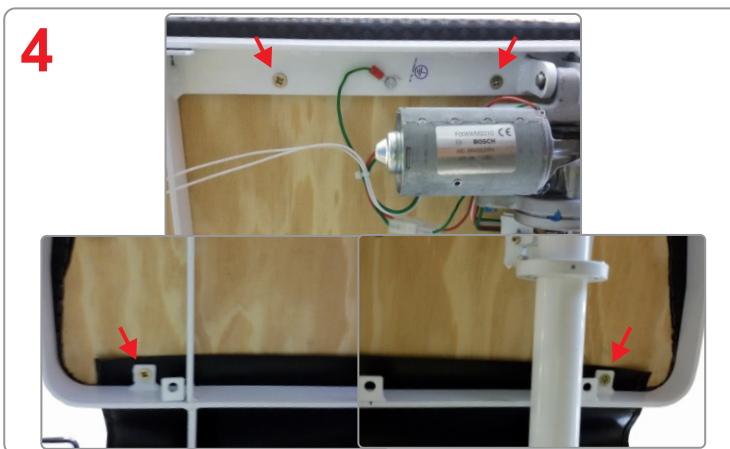


Check the metal frame under the legrest upholstery and find 4 brass philips screws wich hold the legrest upholstery. Remove them.

5

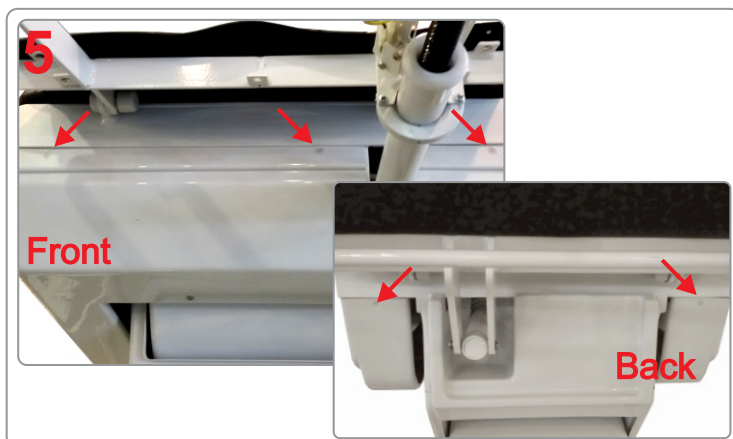


To remove the plastic cover under the seat, find and remove the 5 plastic rivets from the plastic cover. You will reach the motor for seat inclination. Disconnect the black cable from the limit switch, press the seat inclination command for 5s in order to have more seat elevation. Remove the plastic cover and connect the black cable again on the same connector.

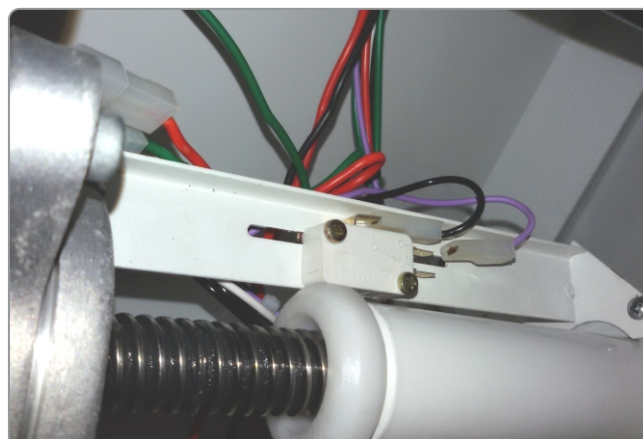





To remove the battery set, lose the screw (A) but don't remove it.  
Remove the battery set from its fitting.

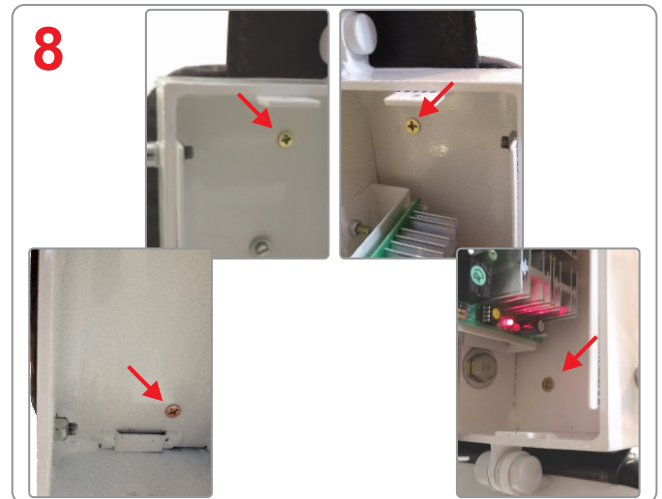

Find 4 brass philips screws wich hold the seat upholstery and remove them.

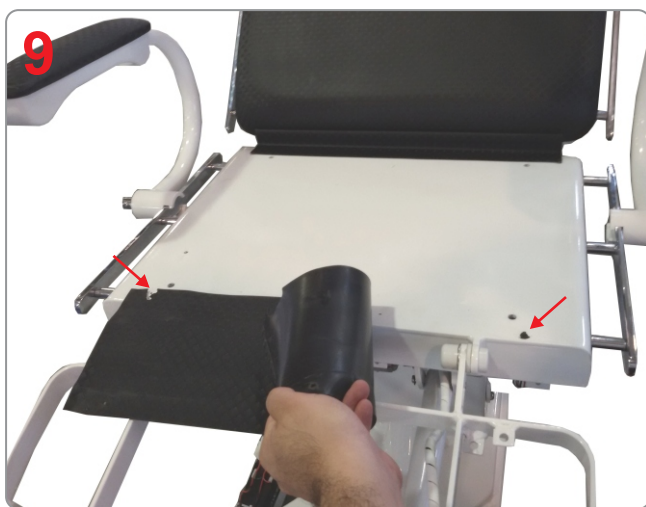

Remove this piece of PVC coat, and carefully remove remaining material from metal frame.

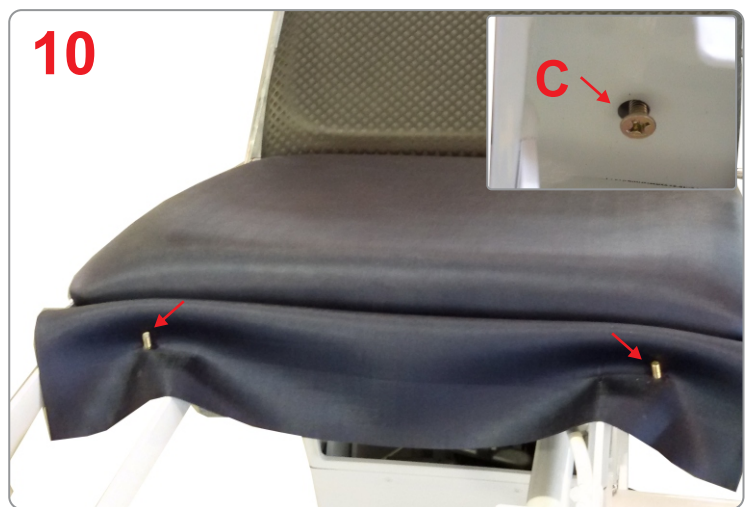

Place new seat upsholtery and put the 4 brass philips screws only using hands (C). After place all screws, tighten them. Place the screws for legrest at the metal frame passing through the PVC coat holes.

**11** Place the legrest, and put the 4 screws only using hands, then tighten them.

**12** Place the battery set and tighten the screw for its fixation.

**13** Place all the plastic covers for seat first, and after for legrest. Use the plastic rivets for fixation.

**14** Press Zero Position command to finish the procedure.