



OPTIONAL WATER VALVE FOR BOWL AND CUPHOLDER


Dear technician,

We inform you that as of 05/09/2022, due to the replacement of the Thermoval solenoid valve by the Emicol solenoid valve, the water valve that controls the water flow to the bowl and cup holder is no longer a line item. This way, the equipment will have the bowl and cup holder water valve as an optional item.


This change is valid for the models Sprint, Sprint Ambidextrous, Sprint Black, and Quality that have a timed water command. In case the equipment does not have a water valve and the client is interested in installing it, the following codes were created for its installation:

- 979080103** Water Valve Kit in the Bowl
- 979080105** Water Valve Kit in the Bowl (Black)
- 979080106** Water Valve Kit in the Bowl and Cup Holder
- 979080107** Water Valve Kit in the Bowl and Cup Holder (Black)

FOR EQUIPMENT WITH TIMED WATER COMMAND

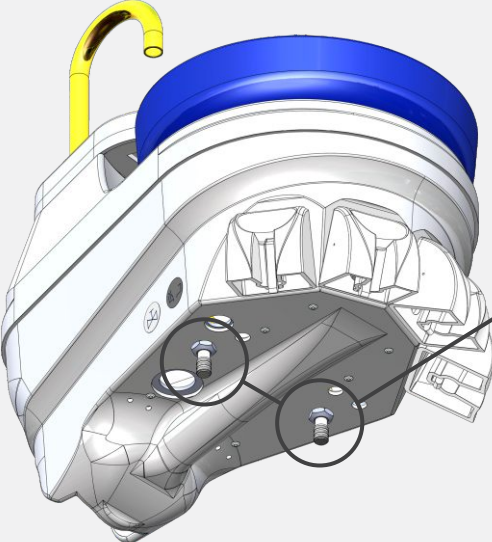


Optional

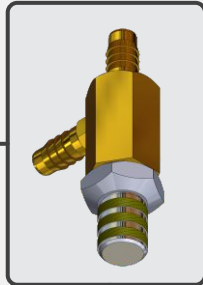

9400077


Complete Water Valve

SIDE VIEWS

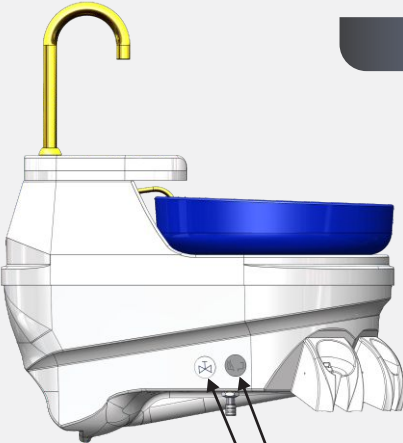

5400120
5400320

Adhesives - Valve and Water in the bowl

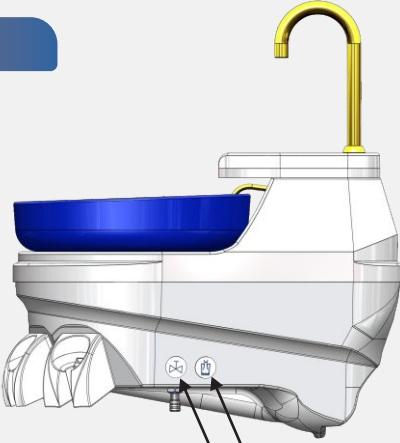

5400126
5400320

Adhesives - Valve and Water in the cup holder

Merely illustrative images

Olsen Tec is an exclusive Olsen newsletter for the accredited technical assistance network, aiming to inform changes in products or procedures, preparing technicians for market innovations.
Pay attention to the content of this newsletter and forward this information to your technical team.

OLSEN INDÚSTRIA E COMÉRCIO S.A.
Av. Ivo Lucchi, 68, Distrito Industrial
Palhoça/SC - Brasil - CEP 88133-510
Phone: + 55 (48) 2106-6000
www.olsen.odo.br